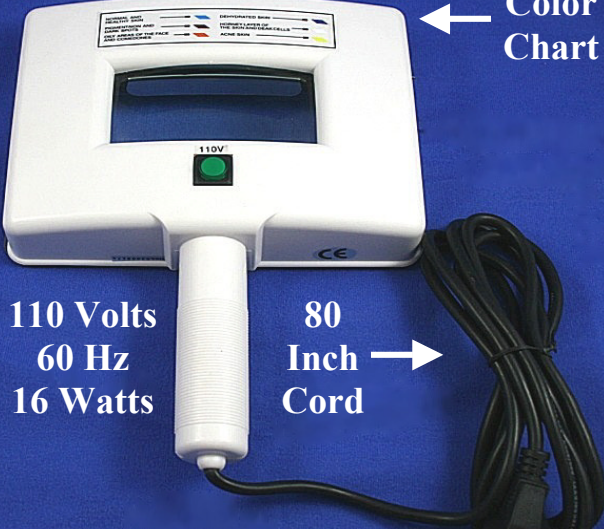


Wood's Lamp TP-8102

 <p style="text-align: right; margin-right: 20px;">← Color Chart</p> <p style="margin-top: 100px;">110 Volts 60 Hz 16 Watts</p> <p style="margin-left: 150px; margin-top: 10px;">80 Inch → Cord</p>	<p><u>FUNCTION:</u></p> <p>The Wood's Lamps main function is to detect items that fluoresce.</p> <p>The Wood's Lamp is used as a diagnostic tool (inspection and detection) – but is NOT a therapeutic or treatment device.</p> <p>The Wood's Lamp is used Worldwide in the following fields: Esthetics, Medical, Law Enforcement, Veterinary Care, Day Care and Schools.</p>
---	---

PRECAUTIONS:

- 1) Clean the client's skin surface with a Rotary Brush before using the Lamp.
 - See Keller International Item # TP-J1004.
- 2) Examine the client's skin from a distance of 5 inches to 20 inches.
- 3) The examination room should be perfectly dark, preferably a windowless room or a room with black occlusive (closed) shades.
- 4) The examiner's eyes should be Dark Adapted in order to see contrasts clearly.
- 5) Do Not Look at the Black Lights for long periods of time.


NOTE: Common sources of error are bluish or purplish fluorescence produced by ointments on the skin-containing petrolatum, green fluorescence by salicylic acid containing medicaments on the skin and light reflected from the examiner's white coat producing a light blue fluorescence.

OPERATION:

- 1) Plug the unit into a standard wall outlet and press the green "ON" button.
- 2) Allow the 4 black lights to warm up for at least one minute.
- 3) Attach the elasticized Black Light Blocking Drape.
- 4) Examine the client's skin and compare against the Color Chart.

NOTE: Slide the clear plastic lamp protector "OFF" of the back of the Wood's Lamp before beginning the examination. Three (3) Diopter Magnification.

Color Chart

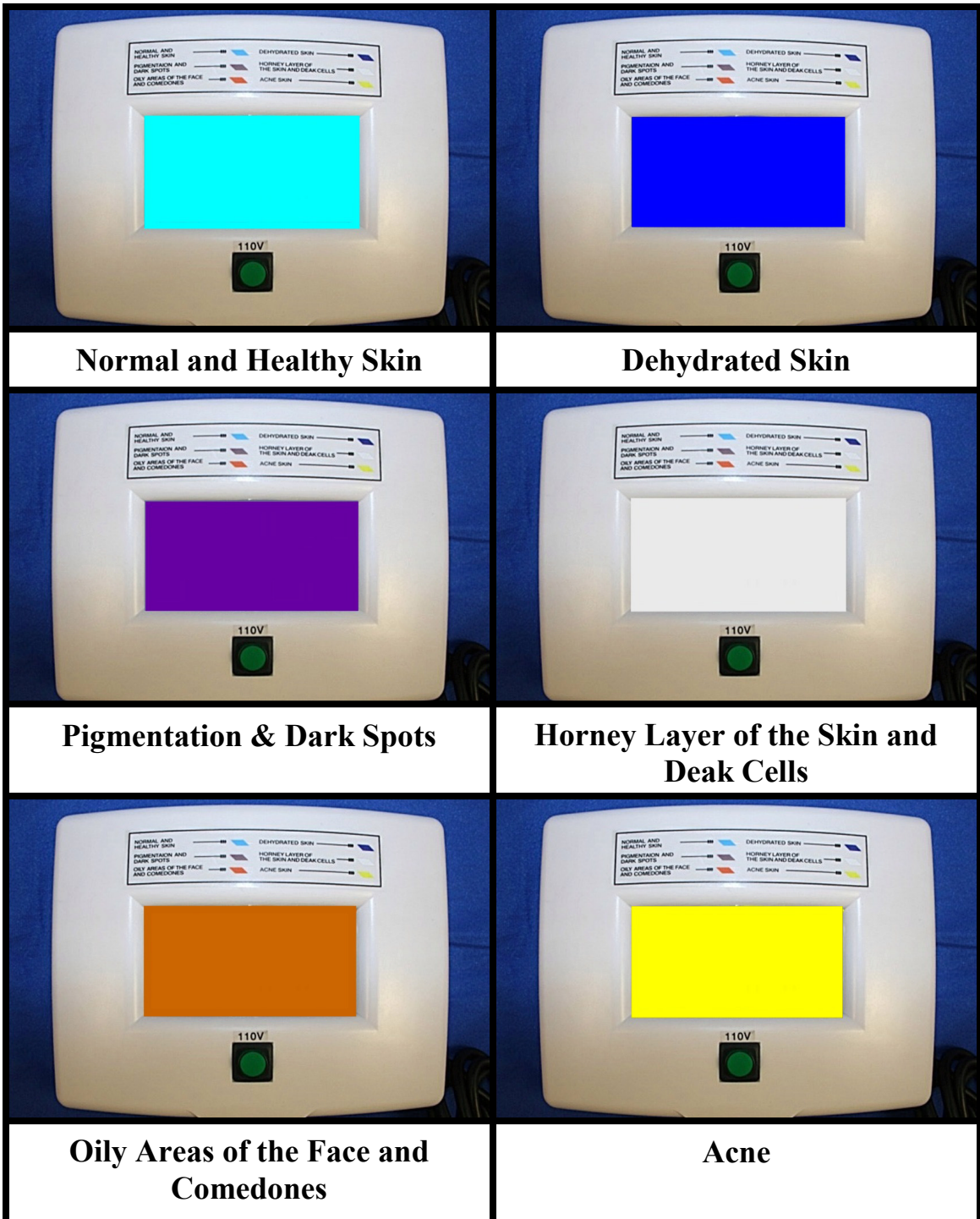

**Deak Cells = Little Boxes or Chambers.
Comedones = A thickened secretion plugging a duct of the skin (blackheads).**

NOTE: THIS COLOR CHART IS ONLY A GUIDE. Keller International, LLC assumes NO RESPONSIBILITY for misuse of this device and assumes ALL Purchasers of this device are Licensed Estheticians or other Professionals. Keller International would advise all potential purchasers to search the Internet &/OR Published Test Books for more information on the use of the Color Chart.